



## USE OF A TRANSFER BELT FOR TRANSFERS

**Transfer belt is positioned snugly around client's waist:**

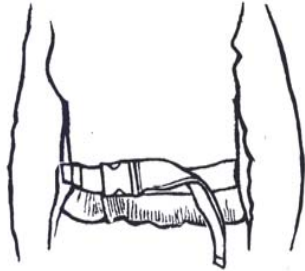


**Correct position**  
Normal use

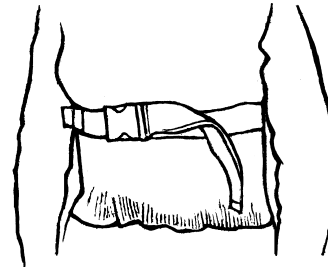

**Use here only if client has colostomy bag or is pregnant.**

**Transfer belt used for transfer and/or walking:**

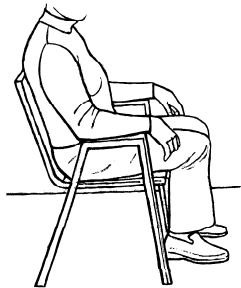


**Client moves hips forward in chair.**




**Care provider puts T. belt around waist and encourages client to push down on armrests of chair.**



**OR**


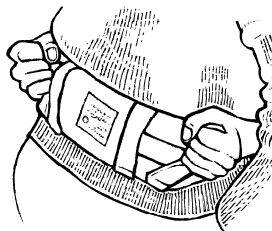


**Care provider grasps loops, pulls and holds client towards self. Care provider stands up drawing client up into standing position.**

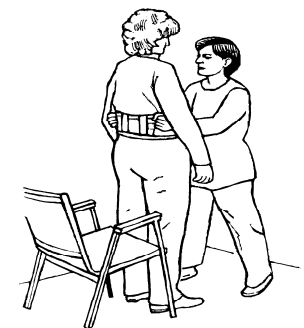

## USE OF A TRANSFER BELT FOR ASSISTED WALKING

Transfer belt used for transfer and/or walking continued...

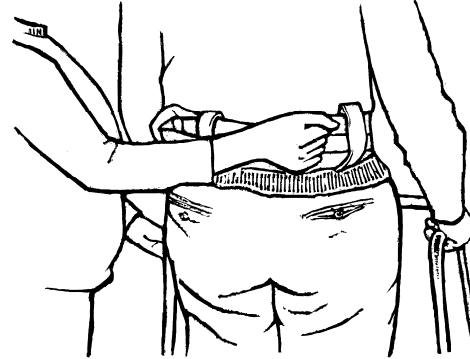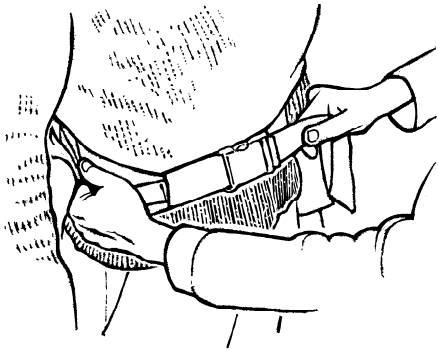


Upon standing, belt will need to be tightened.

**PLEASE REMOVE TRANSFER BELT AFTER TRANSFER OR WALK.**

**DO NOT:**

Put belt over someone's head

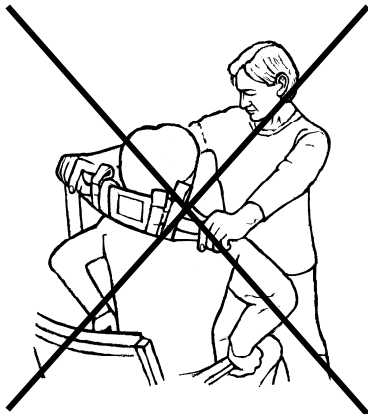

OR

Thread your hand or forearm through straps

